PRIME IMPLANT

Zahnimplantatsysteme

MOMENT
MEDICAL 

PRIME IMPLANT

Dental Implant Systems

MOMENT
MEDICAL 

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TABLE OF CONTENTS

About Us.....	09
Mission&Vision.....	11
Why Prime Implant.....	13
Implant.....	15
What is Dental Implant?.....	17
Surface Treatment.....	21
Titanium Grade 4.....	23
Application.....	25
Resistance Test.....	27
Implant Series.....	29
Prosthetic.....	37
Multi Abutment.....	53
Multi Abutment Prosthesis Applications.....	59
AIPConcept.....	61
Prosthesis Application.....	65
Surgical Set.....	67
Surgical Application.....	71
Implant Application.....	73
Step by Step Application.....	75
Communication.....	77

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MOMENT
MEDICAL 

ABOUT US

Investor partners of Moment Medical, a German company, have been in the health care sector since 2008 with activities for patient care in Germany. The Moment Medical operation, which we started with our work that creates added value for the health sector in Germany, aims to create a major change in the field of healthcare with excellence in patient relations, and a healthy operation carried out in partnership with insurance companies.

In all the commercial activities we have carried out so far, we have aimed to differentiate with quality, corporate reputation, and the economic value we have revealed. The foundation of our investment in quantitative and qualitative quality is people and the importance we attach to them.

We believe that the greatest advantage of Moment Medical's success in the field of healthcare is quality and trust, which is part of our corporate culture.

We are proud to add dental implant systems developed with superior German technology to our investments in the field of health. It is our greatest wish that our investments, which will be an essential locomotive for our international R&D activities, reach all people.

Moment Medical GmbH
Board of Directors

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MISSION

In order to achieve moment medical vision:

- We will offer innovative products and services that will increase quality in the field of health to the service of the society in the most appropriate conditions.
- We will address our vision in the field of health within the framework of corporate social responsibility by supporting academic studies.
- We will develop healthcare know-how by enabling our doctors to cooperate and communicate.
- We will carry out all our work in accordance with sustainability activities.

VISION

To be a reliable bridge on the road to health for the people of the world, to improve the quality of life and to create a reference service framework by maintaining innovative investments.

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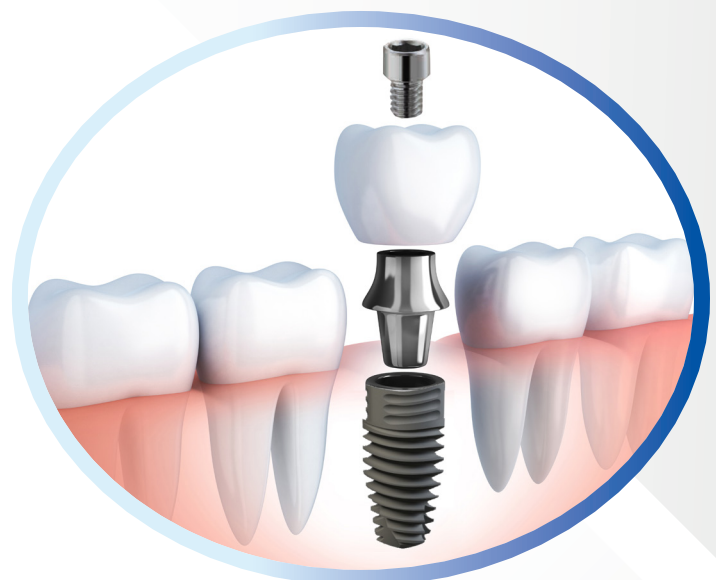
WHY PRIME IMPLANT?

AIP (ALL INCLUSIVE POLICY) CONCEPT

Parts of implants used today, such as healing cap, analogue and impression post, are the parts that are transferred from patient to patient. Disadvantages of part transfers are cross contamination, infection due to lack of sterilization, difficulty for doctors to reach the necessary parts on time and the necessity of bearing extra costs. The aim of AIP CONCEPT is to eliminate these disadvantages. There is a single-use healing cap, analogue and impression post on the AIP carrying part. These personalized parts are sterile and in the same packaging as the implant. We can use the impression post as a temporary abutment. Since analogues are personalized, plaster casts can be stored as long as desired. With AIP CONCEPT, the stocking cost is eliminated.

WTIS (WATER TEXTURE IYONIZED SURFACE) SURFACE TREATMENT

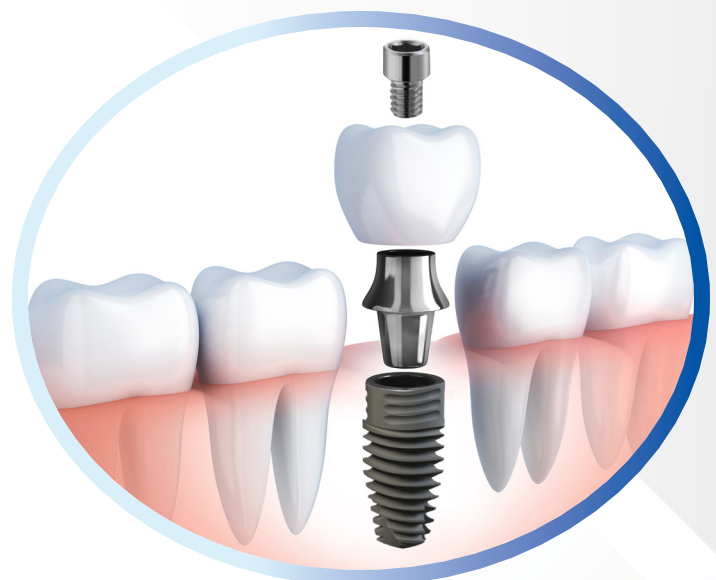
Acid, which is a hazardous waste, must be removed from the rough implant surface to maintain a clean surface. If acid purification, which is a multi-stage and sensitive process, is done incompletely, the success of the implant is at risk. The surface of the PRIME implant is activated by hybrid sandblasting +pressure washing and UV treatment to obtain a reliable, clean and standard surface.

LOCKING


The most important junction area of the implants is the abutment. Design errors in this area cause many undesirable conditions such as screw loosening in long use, loss of supporting bone because of the loosening and even implant fractures. For this reason, the stress caused by chewing should be evenly distributed with deep locking. Today, deep deadlocks are steps taken in this direction. But the ideal is a deep conical lock. Mechanical studies have determined that the deep and conical contact of the screw with the abutment is the ideal combination to create a more durable leak-proof locking and the locking has been designed in the light of this information

SPECIAL BUR DESIGN

Since the implant burs are designed at special angles, it eliminates the need to use neck expanders in all bones. Thus, in implant applications, the process is completed with a final drill, thus shortening the surgical time and providing great convenience.

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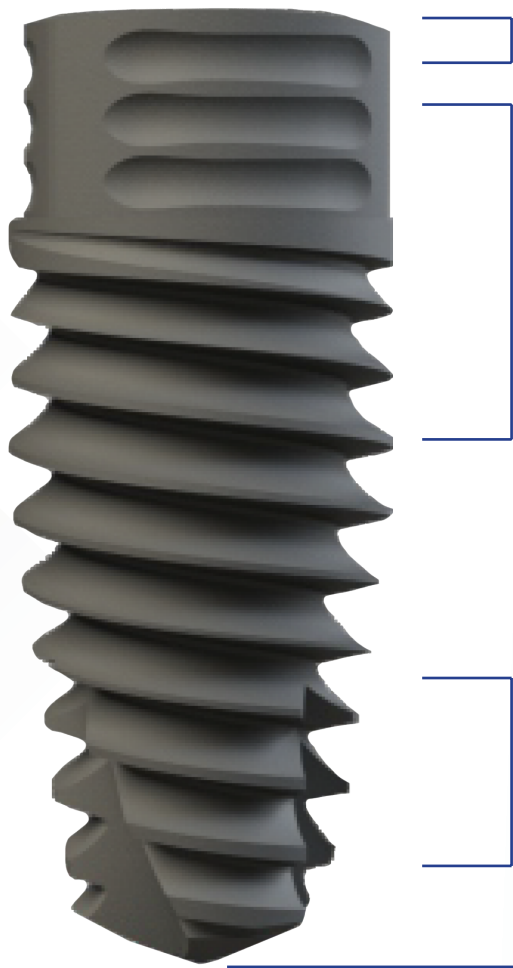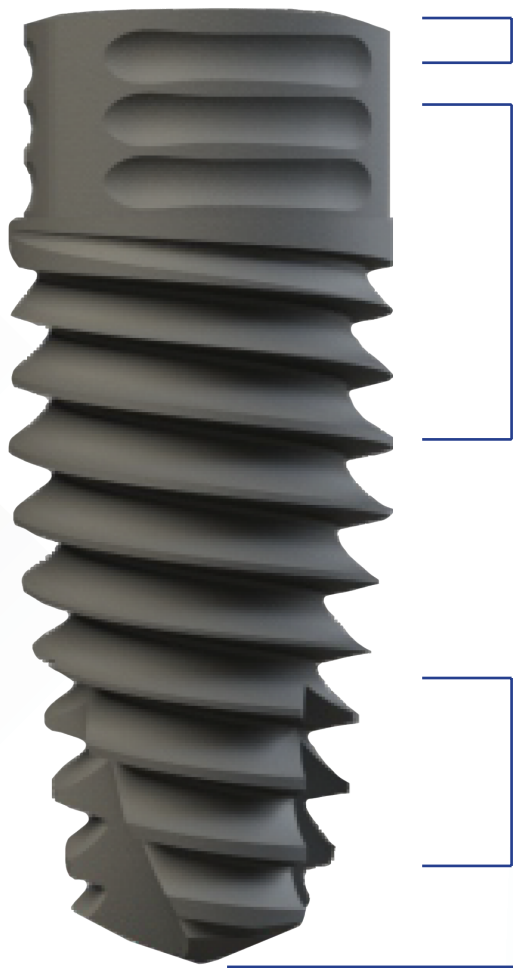
IMPLANT

Platform Switching
Compression Area

Thread and diameter differences reduce the stress on the bone. The blood pool accelerates healing and ensures bone compression.

The threads deepening towards the tip trap the patient's blood during the surgical phase and accelerate healing.

Rounded bottom tip to protect sinus floor and sinuses

PRIME implant has a cylinder and special thread structure that can be mechanically placed safely in every clinical situation and is designed to distribute the load evenly.

It has been designed to offer ease of use with a high success rate in cancellous, Type 3 and Type 4 bone, thanks to its rounded lower tip suitable for all types of implant surgery.

- Platform Switching
- A Good Stability
- Specially Designed Thread Structure
- Cylindrical Root Shaped • Rounded Bottom End
- Compression Zone
- Perfect Fit With All Abutment Systems

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WHAT IS DENTAL IMPLANT?

WHAT IS DENTAL IMPLANT?

It is an artificial tooth root made of tissue-friendly materials in order to restore the functions of the missing tooth or teeth and their contribution to aesthetics and speech.

HOW IS DENTAL IMPLANT TREATMENT PERFORMED?

In order for the implants to be applied, the quality of the jawbone in the predicted area, together with the appropriate height and width, is important.

Implants can be successfully placed in areas with healthy bone and mucosa with very simple procedures.

Healing and the application of the superstructure are determined by the doctor according to the application area.

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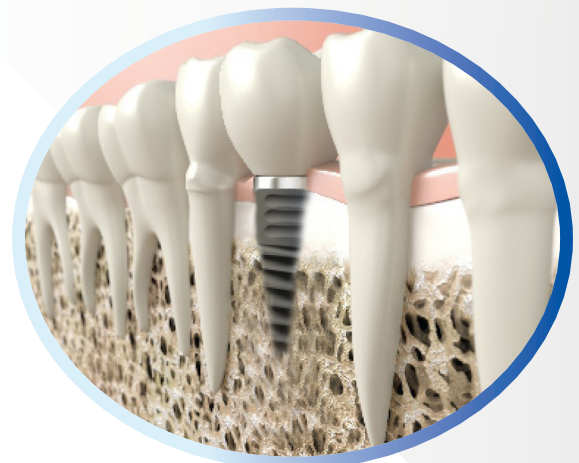
IN WHICH SITUATIONS IMPLANT IS APPLIED?

In there is one, multiple or completely lost teeth, implants replace these losses.

In the absence of a single tooth, treatment is carried out without the need for neighboring teeth.

In the treatment of multiple or completely lost teeth, aesthetic problems, chewing mobility, food leakage under the prosthesis, difficult cleaning, frequent renewal and inappropriate pressures are eliminated with implant supports.

WHAT ARE THE CONDITIONS WHEN IMPLANT TREATMENT IS NOT POSSIBLE?

Implants can be applied easily, except for uncontrollable systemic disorders and the unsuitable amount of bone quality in the jaw.

The amount of bone can be made suitable by additional surgical methods.

IS THERE A RISK OF IMPLANTS NOT BEING ACCEPTABLE BY THE ORGANISMS?

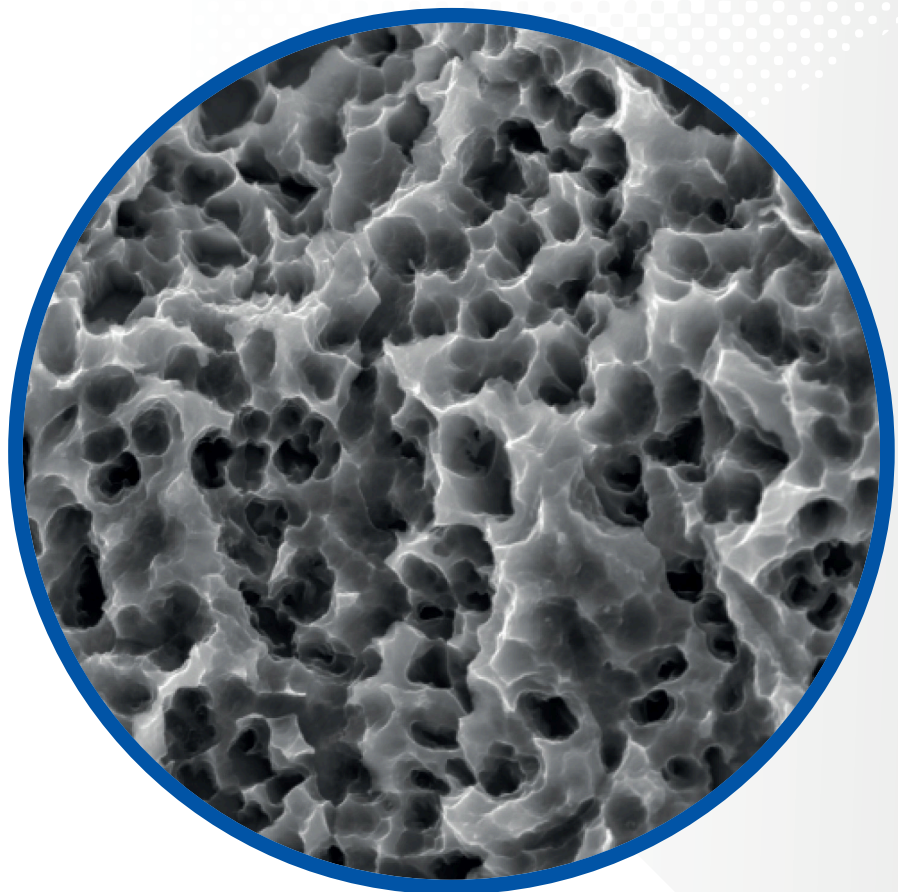
Implants are made of metals such as titanium, which do not have any side effects for the organism and have been accepted after scientific clinical research for many years.

It is not possible for the organism to reject the implants applied with proper planning and surgery.

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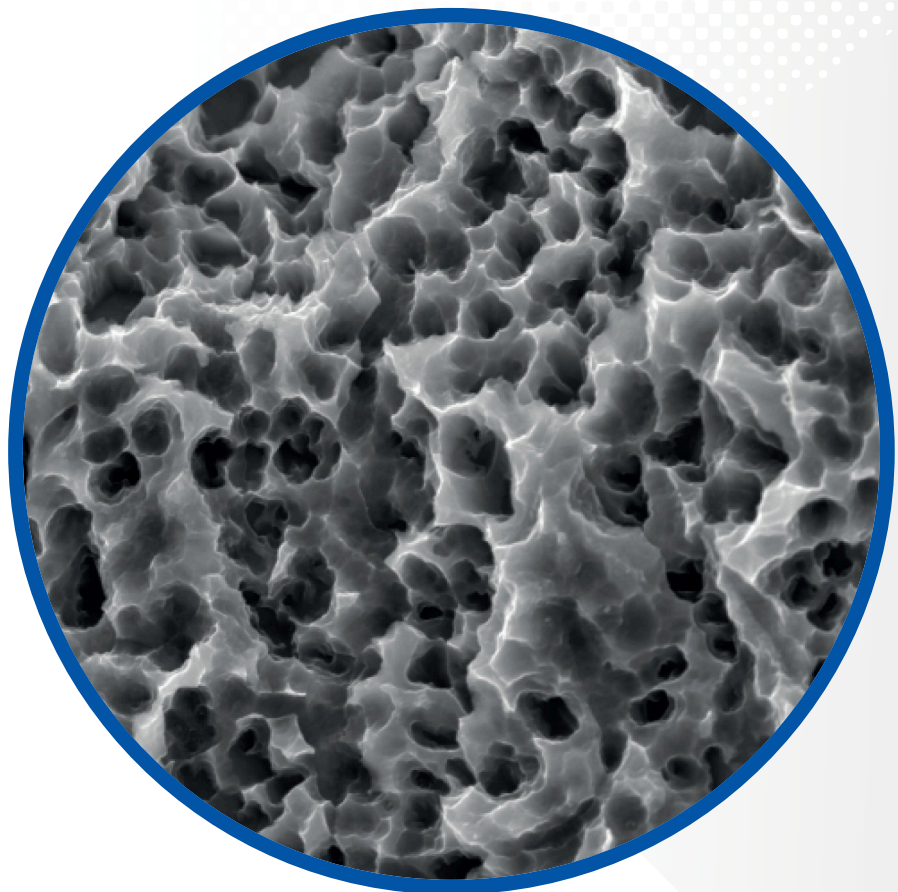
Acid, which is a hazardous waste, must be removed from the rough implant surface to maintain a clean surface. If acid purification, which is a multi-stage and sensitive process, is done incompletely, the success of the implant is at risk. The surface of the **PRIME** implant is activated by hybrid sandblasting+pressure washing and UV treatment to obtain a reliable, clean and standard surface.

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Titanium GRADE 4

Titanium Grade 4 is an unalloyed, high purity grade 4 titanium.

This material provides outstanding mechanical strength and oxidation stability as well as outstanding all-round biocompatibility and anti-allergic properties for the manufacture of dental implants.

Leading global manufacturers use Titanium Grade 4 for manufacturing. It is also referred to as “commercially pure titanium”.

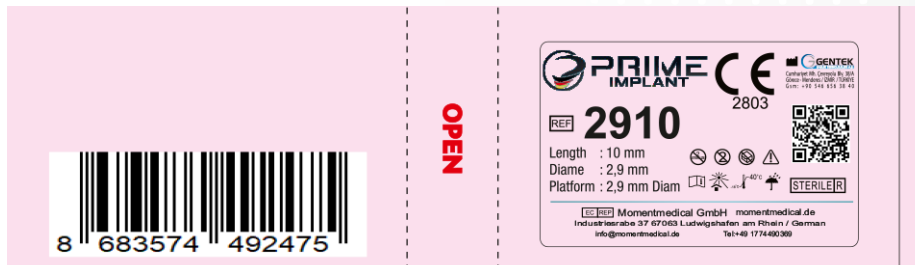
(Standardized by American company ASTM)

Advantages of Titanium Grade 4 over other grades of pure titanium:

- It does not contain toxic vanadium. It is completely safe for implant production.
- It has more durable mechanical properties
- It has better biological compatibility
- It eliminates allergic reactions to %99.9

The fact that Titanium Grade 4 has been standardized for manufacture by ASTM indicates its suitability for dental implants.

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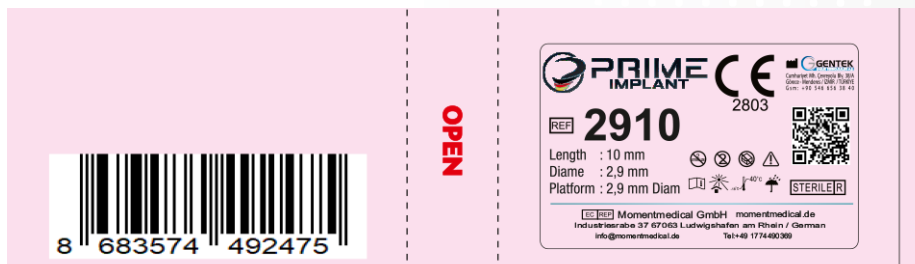
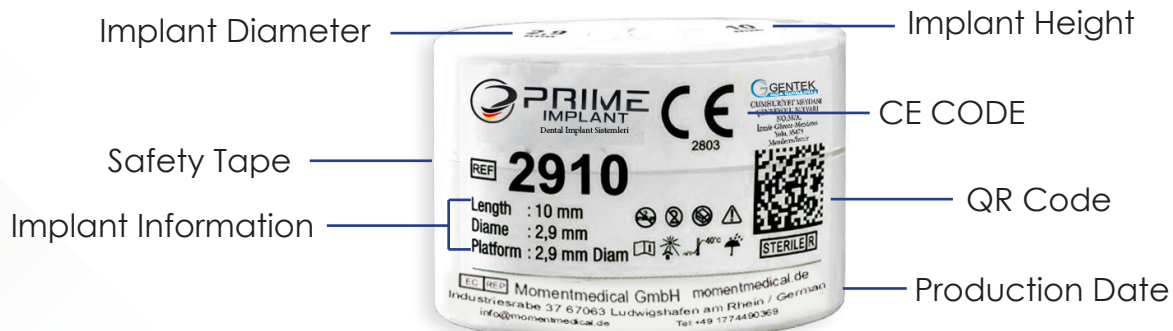
APPLICATION



Open the box to get the implant with the transfer part



Implant cover screw can be taken out with the transfer part

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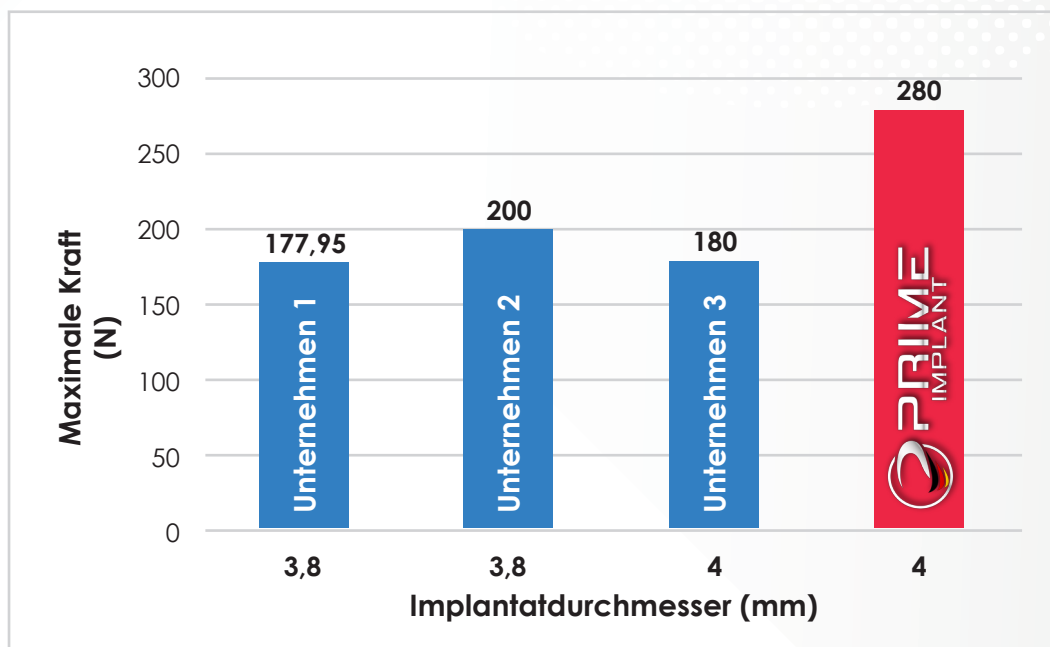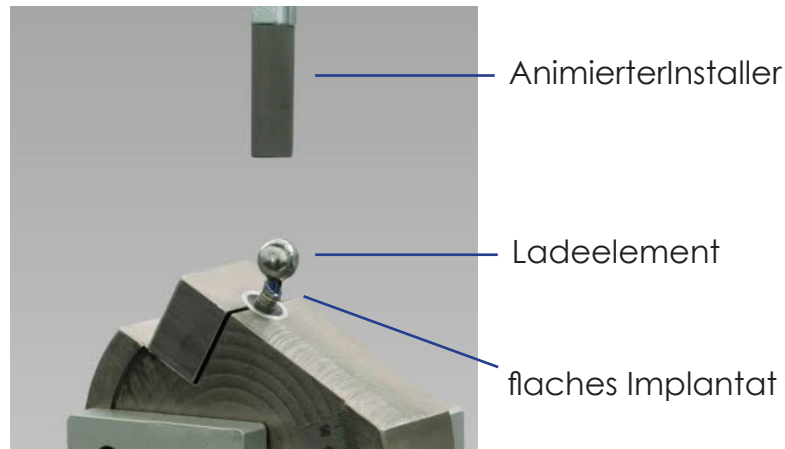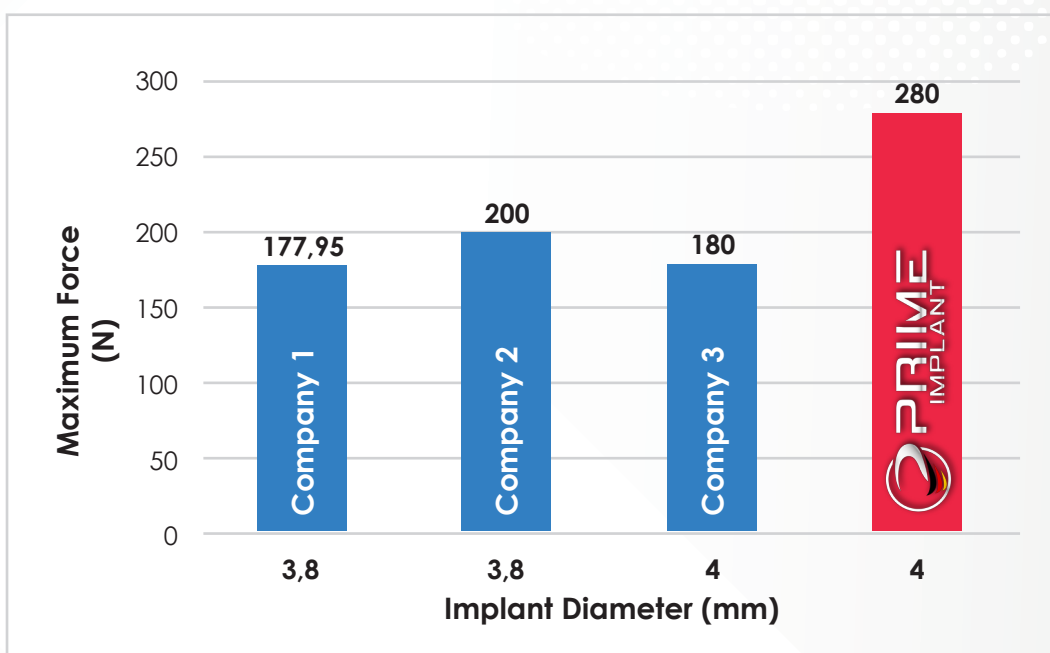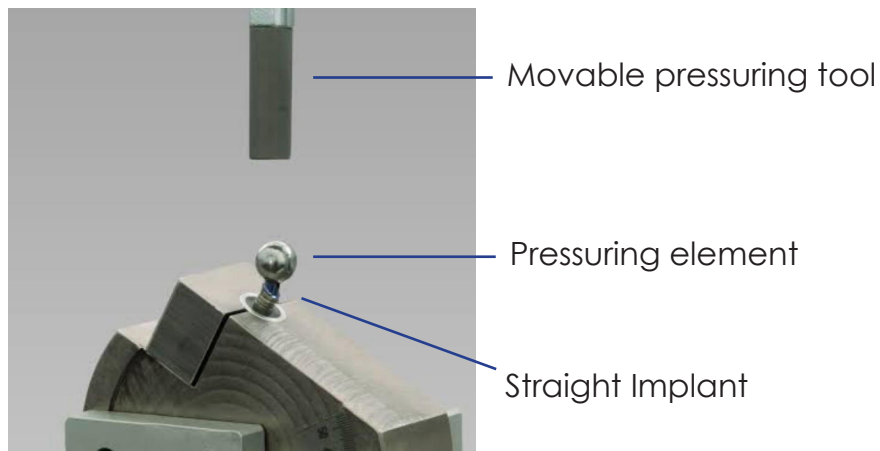RESISTANCE TEST

DATA FROM ACCREDITED INTERNATIONAL RESEARCH CENTER

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


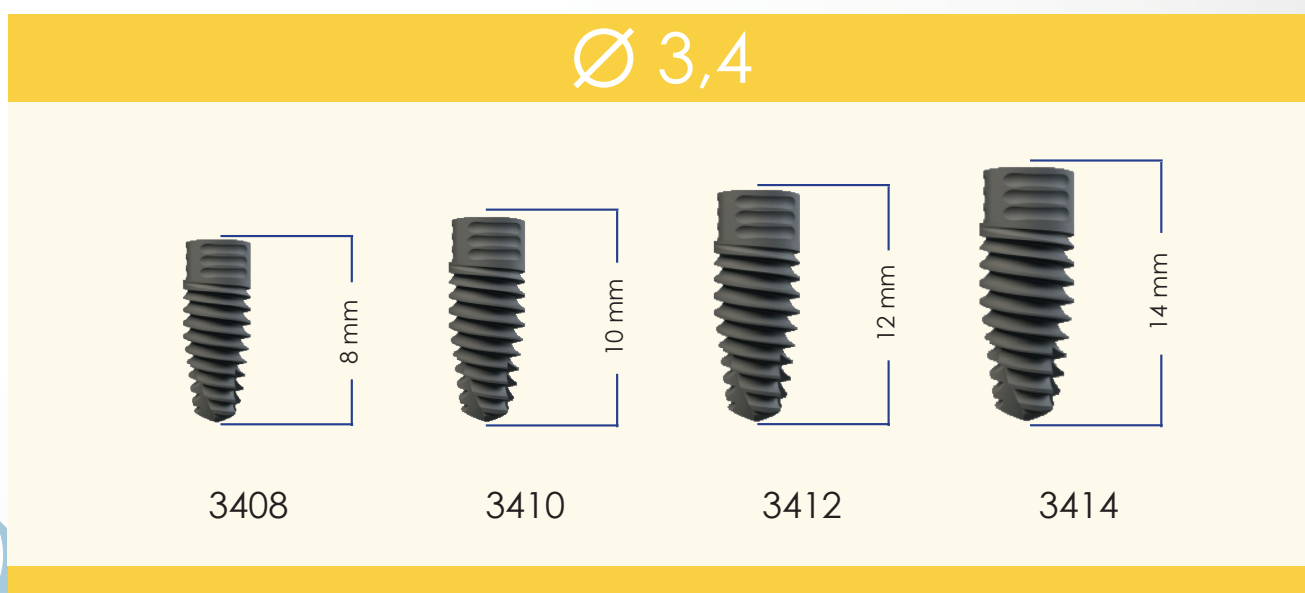
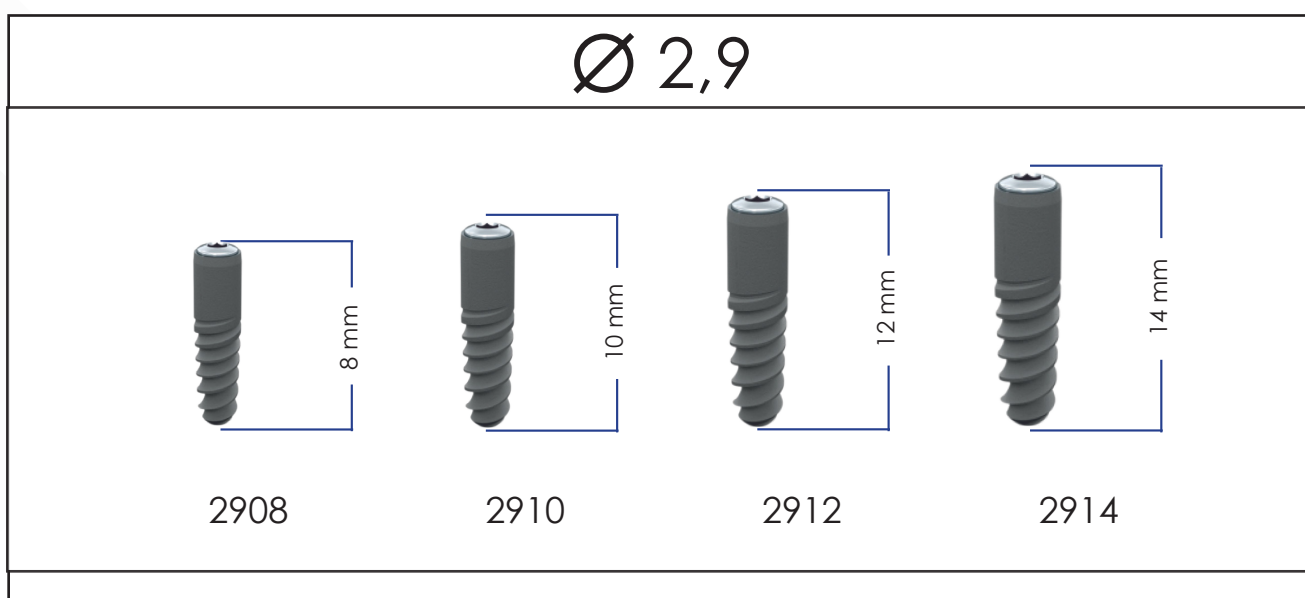
IMPLANT SERIES

Implants are produced as standard in all diameters and lengths that are clinically required.

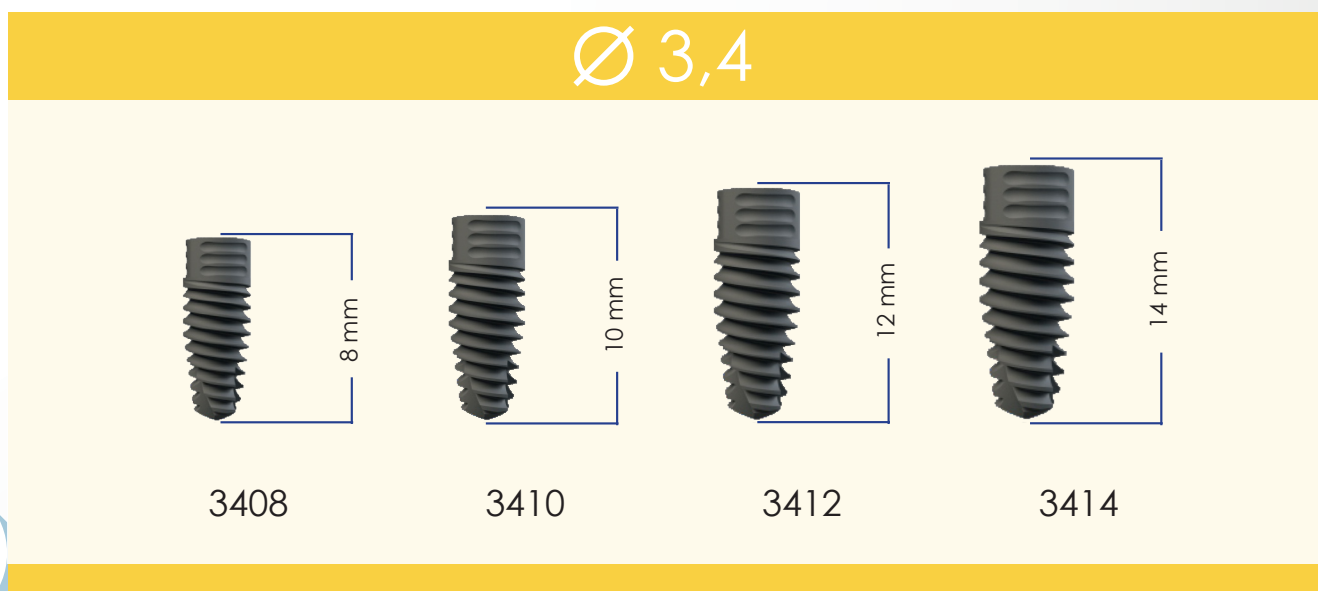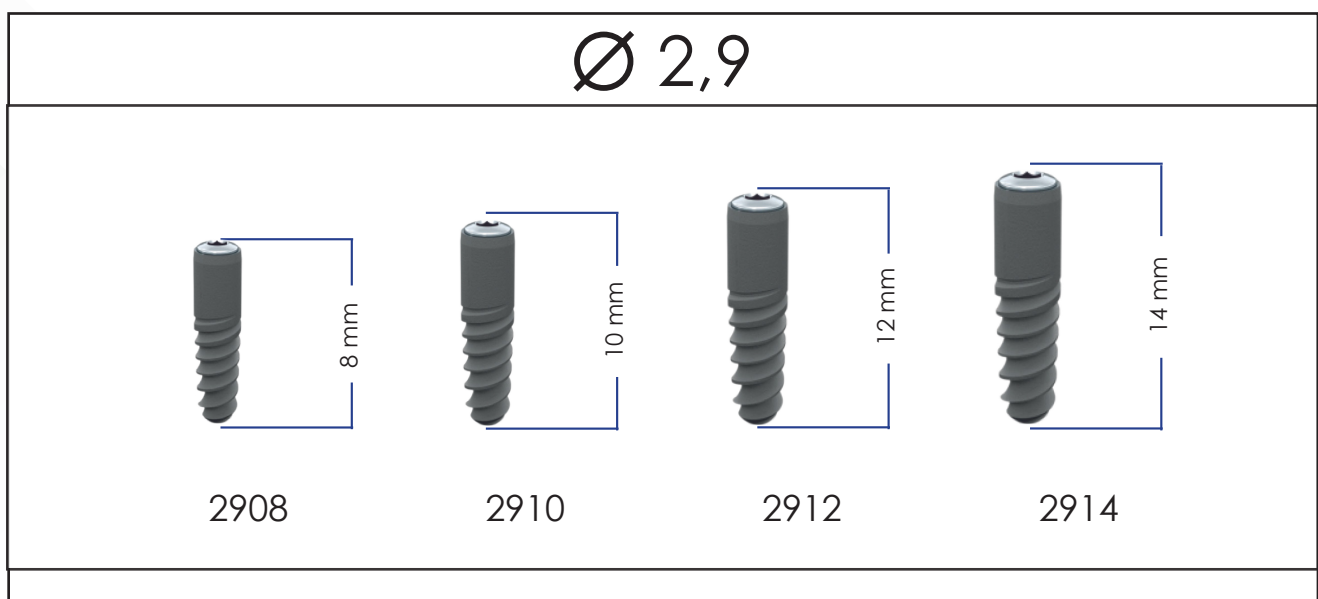The diameters and lengths that are being produced are determined as a result of clinical analysis and are produced according to ISO 13485 standards.

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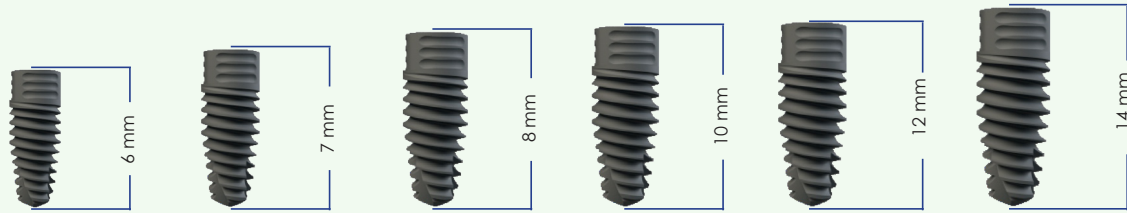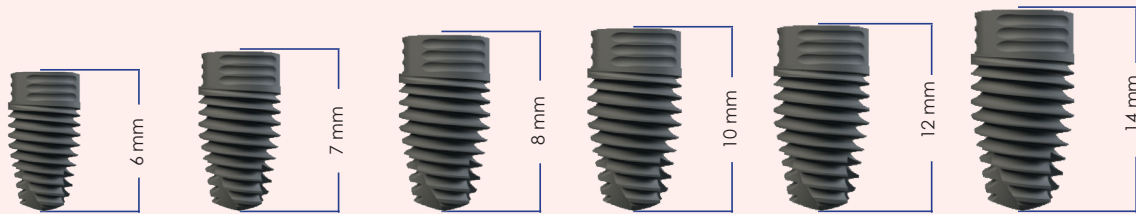
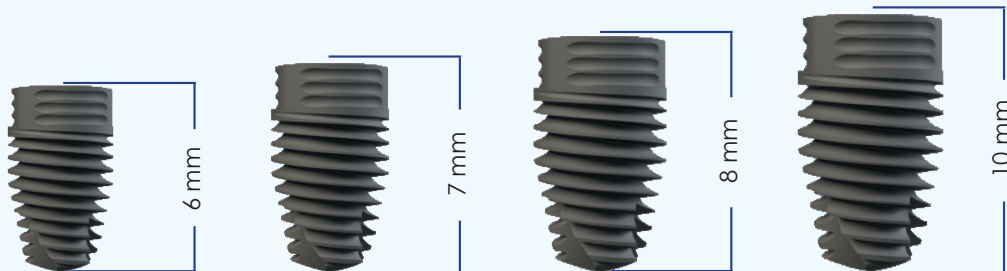
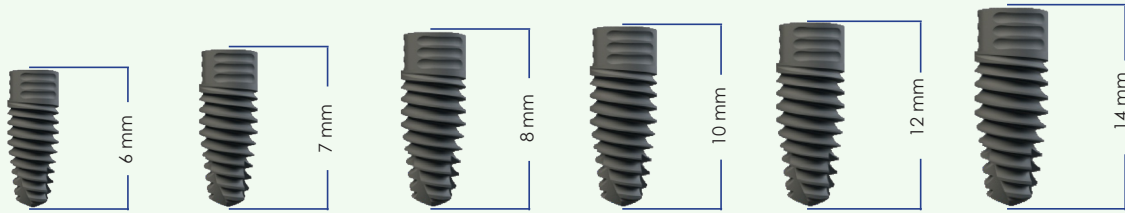
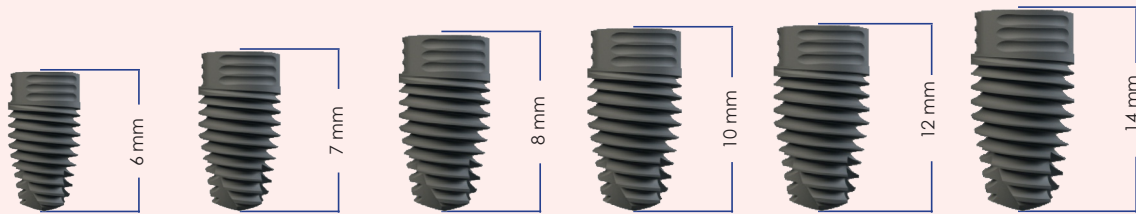
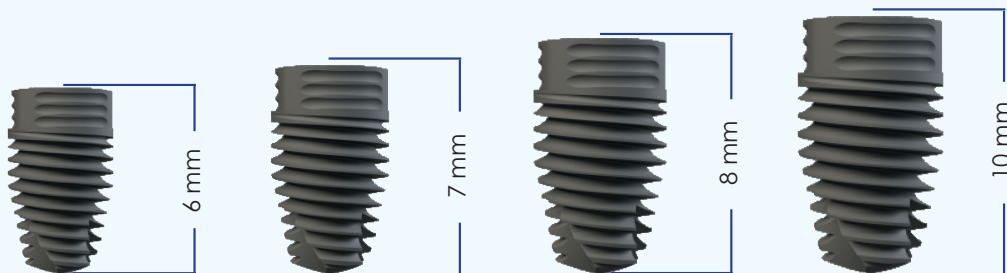
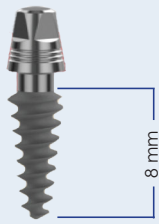
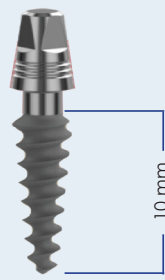
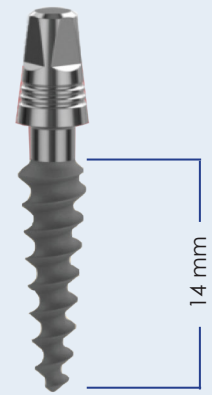
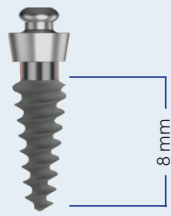
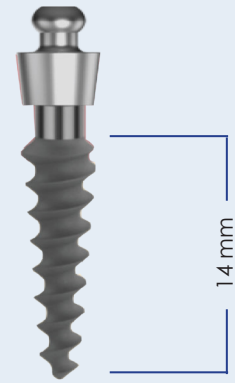
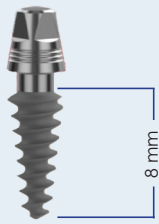
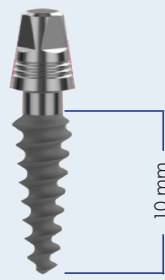
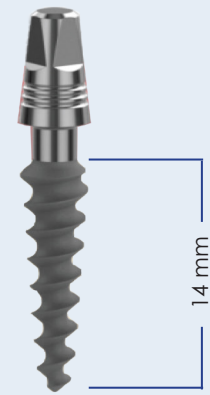
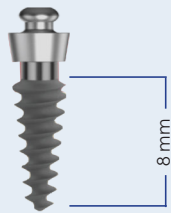
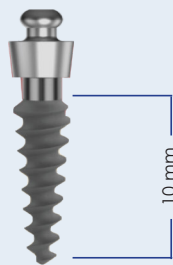
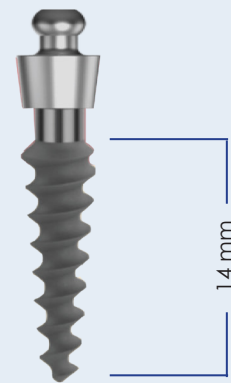
PIB-357

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PROSTHETIC

PRIME is a combination of innovations and technologies, research and experimentation. We compete with world leaders in quality by releasing a product that meets modern implantology trends.

Always perfect results with **PRIME** Implants, which offer a wide range of prosthetic components required for the surgical protocol!

The prosthetic components are highly accurate in manufacturing and parameters. It ensures ideal results.

Prostheses are made of Titanium Grade 5 and come with two screws. We also have unique healing caps in our product range, which provide maximum space for soft tissue. During the healing process, the soft tissues acquire the necessary size and shape. Due to the use of this component, the natural contour and size of the gums besides desirable smile can be achieved

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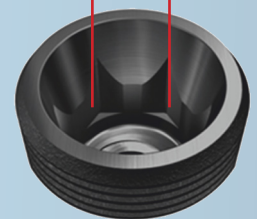
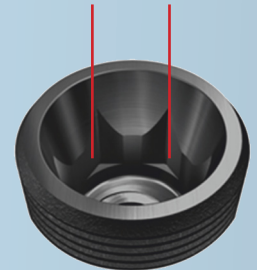
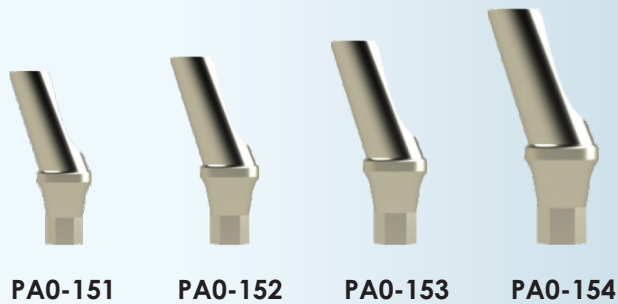
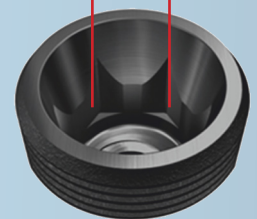
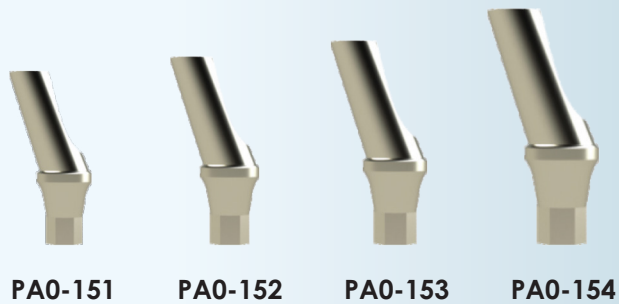
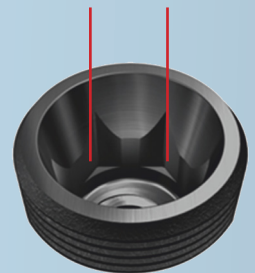
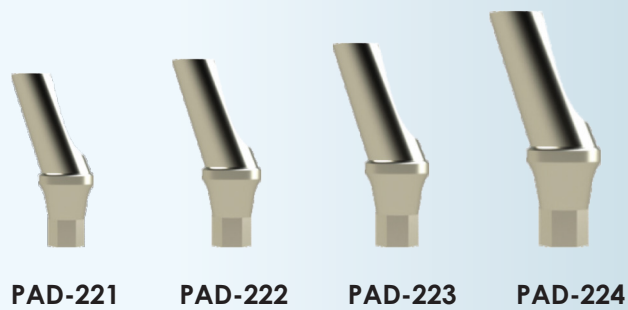
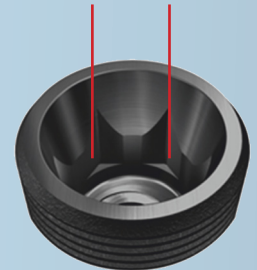
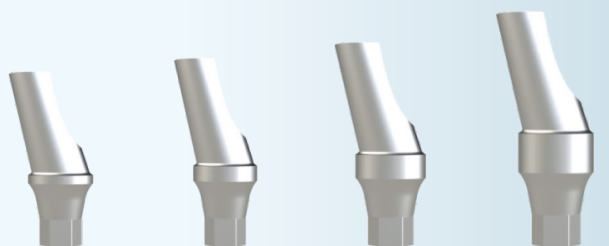
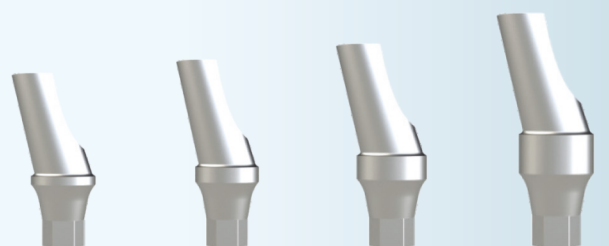
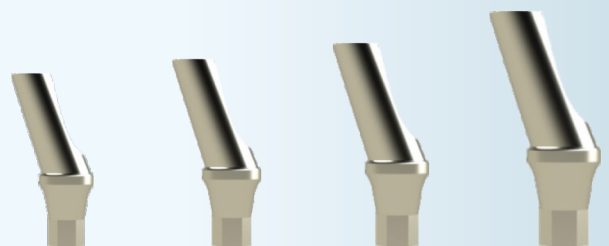
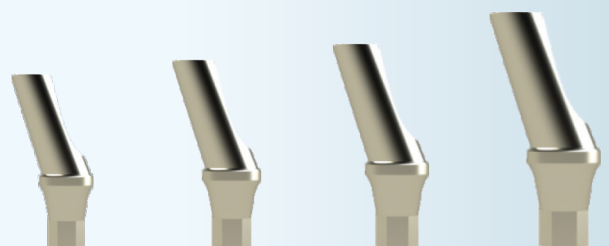
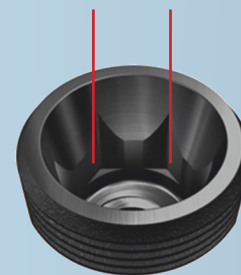
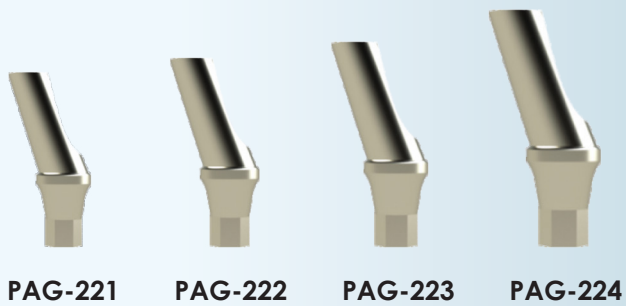
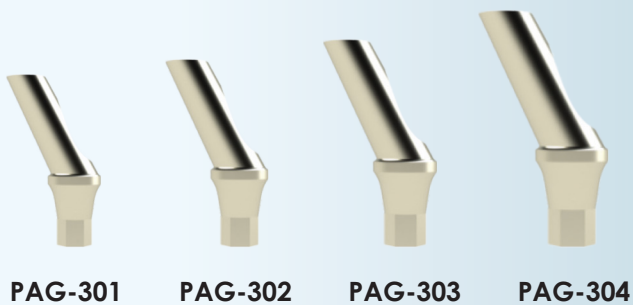
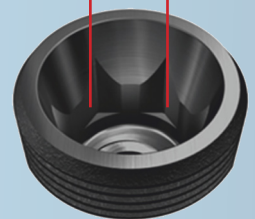
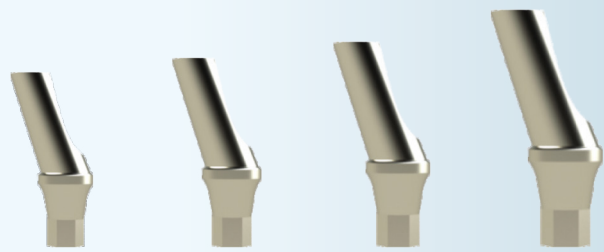
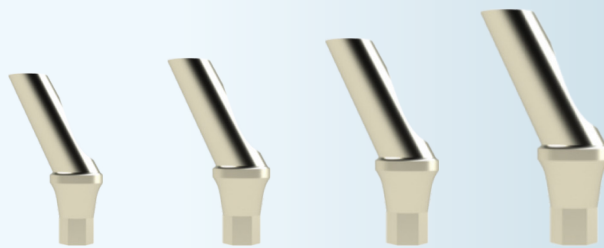
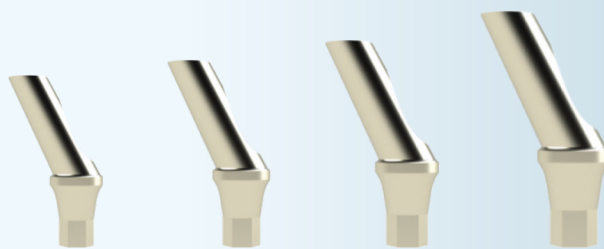
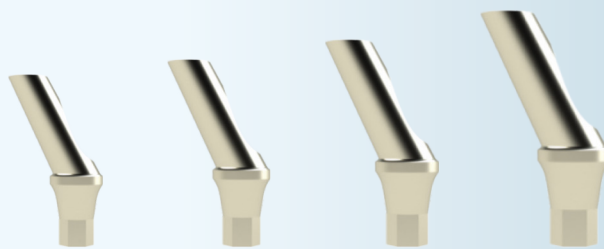
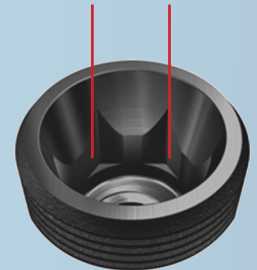
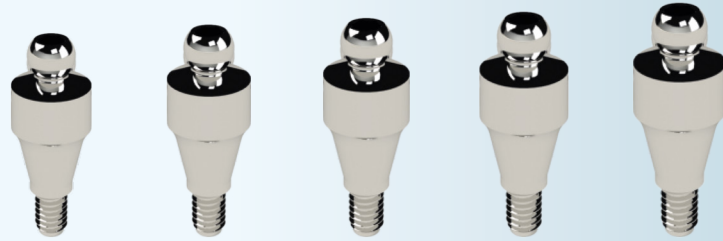
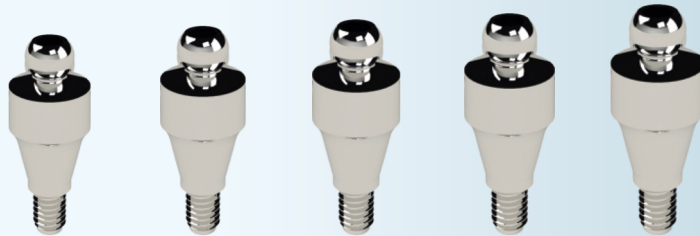
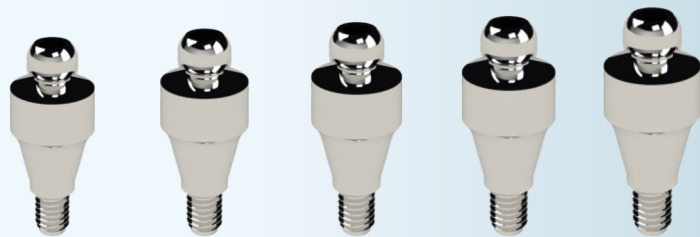
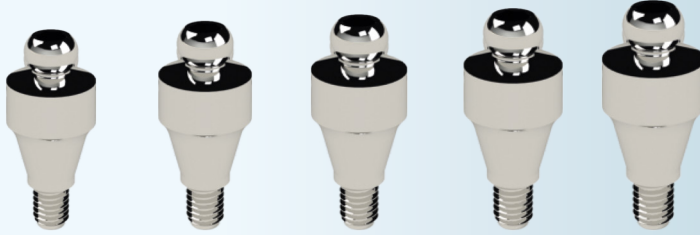
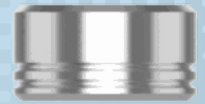
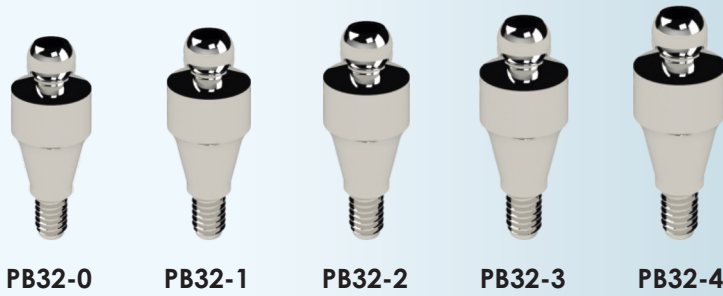
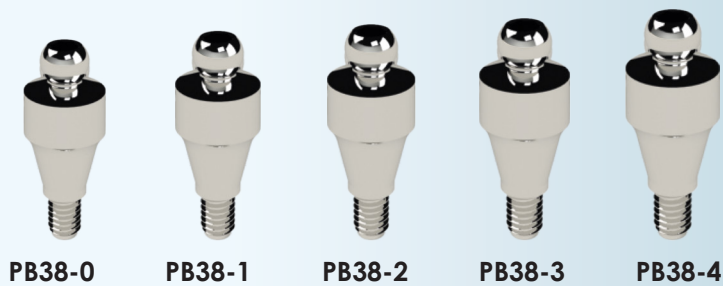
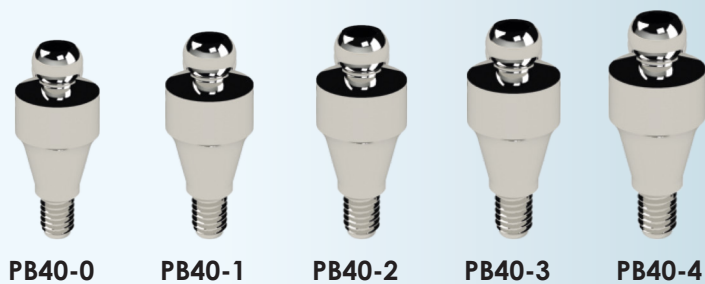
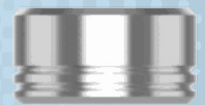
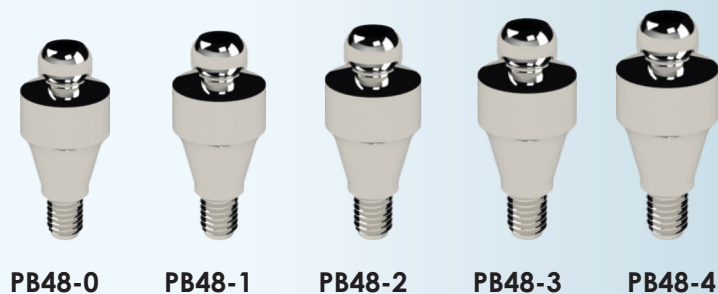
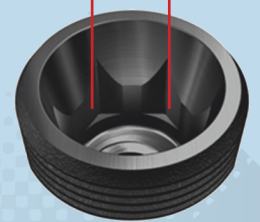
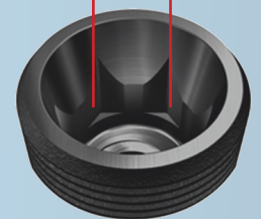
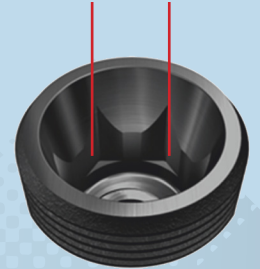
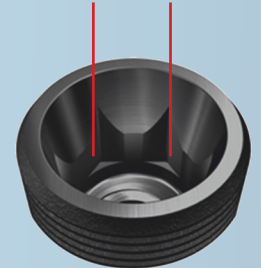
PMS-4

MULTI ABUTMENT

With its multi-unit system, **PRIME** implant offers a wide range of products for dentists and their patients.

Multi-unit abutments provide screw fixation of the denture without the use of adhesives, ensuring the possibility to remove the dental prosthesis when necessary.

Multi-unit abutments are designed for use with prostheses supported by two or more implants.

Along with straight Multi-unit abutments, Multi-unit abutments have a gingival height of 2-4 mm, and the Multi-unit is angulated at 17 and 30 degrees.

Angular Multi-unit abutments have a single prosthetic platform in a conical shape.

We recommend a tightening torque of 20-25 Ncm for fixing the multi-unit abutment and 15 Ncm for the prosthetic screw.

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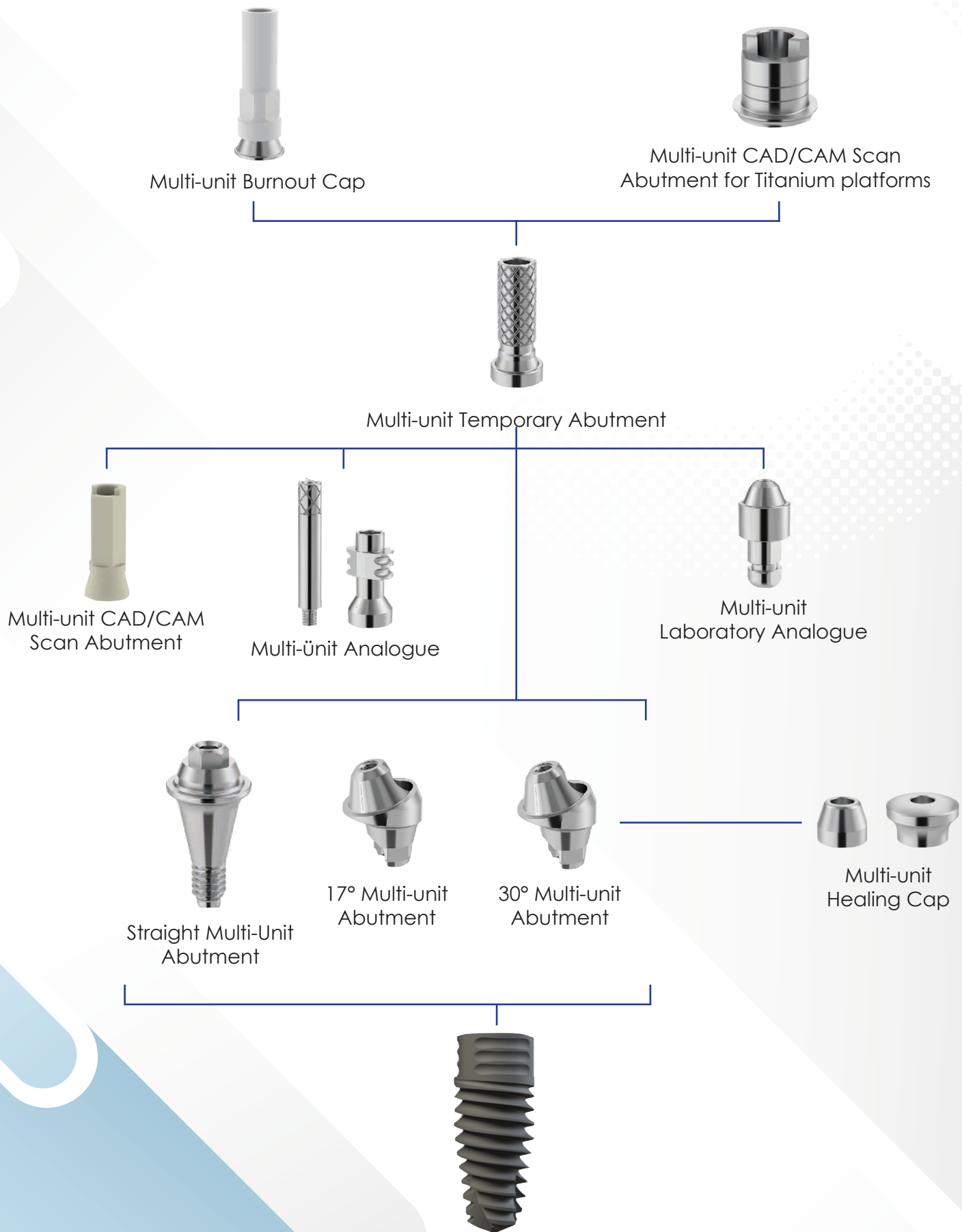
MULTI-UNIT PROSTHESIS APPLICATIONS

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AIP (All Inclusive Package) Concept

AIP means packaging of a healing cap, analogue and impression post for each patient in the same sterile package.

Purpose of AIP Concept:

- To ensure infection control, which has been a problem in dentistry for years and has recently entered our daily life with the pandemic period.
- To reduce the part inventory cost
- To make the material supply as fast as possible

Advantages of AIP concept

Healing Cap

- It can be shortened to the desired extent in patients who are recommended to use a total prosthesis temporarily.
- Obtaining the desired exit profile with composite in immediate surgical applications in the molar region

Impression Post

- It can be used as a temporary abutment when deemed appropriate.
- The impression post can be shortened by cutting it in the neck in case of limited mouth opening.
- Thanks to its design, it does not allow directional errors in transferring from the mouth to the impression.

Angular Impression Post

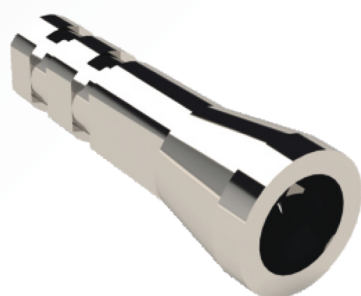
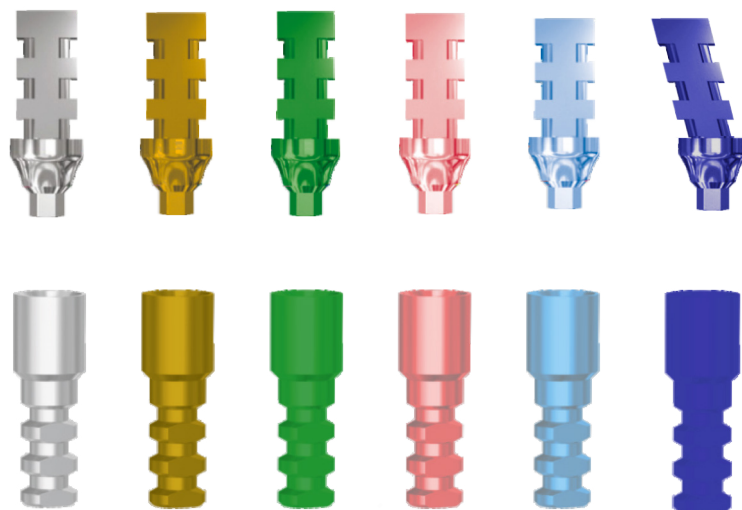
- Contributing to a faster and more accurate measurement of angled implants with the closed impression method

Analogue

- It allows preservation of patient models for a long time.
- It eliminates the need for new impressions in case of prosthetic revision.

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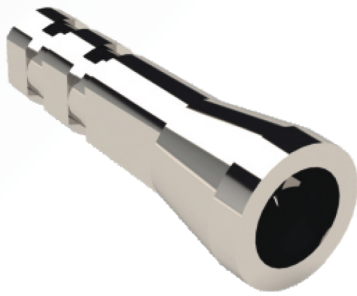
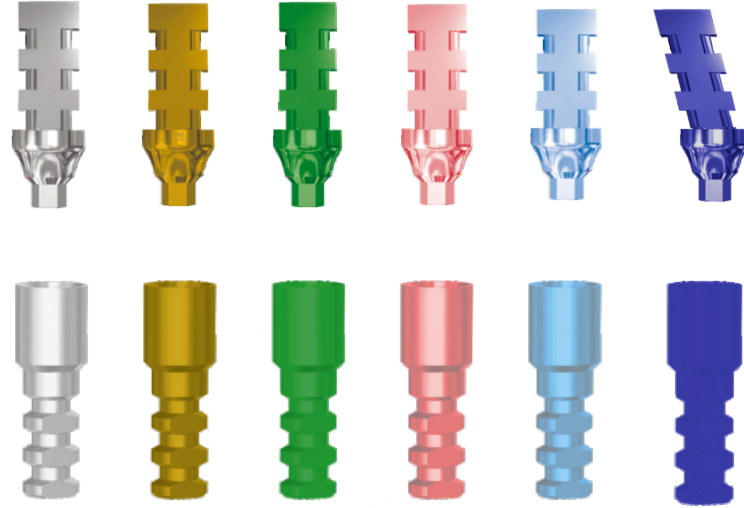
Ø 1,6 mm

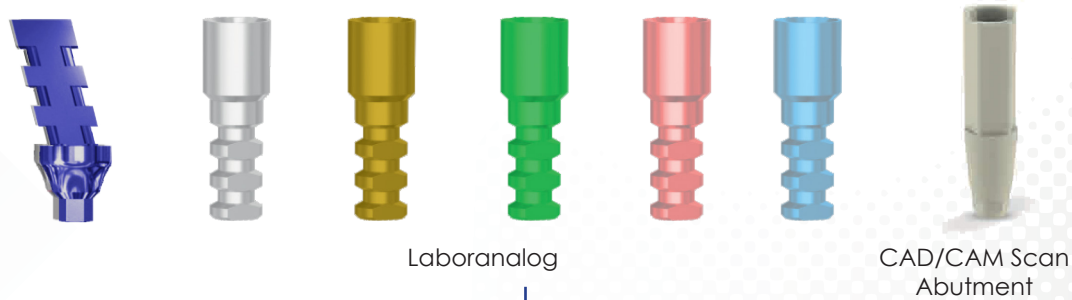
Ø 1,8 mm




It is recommended to apply a Torque of 20-25 Ncm with a 1.25 mm Torque wrench.

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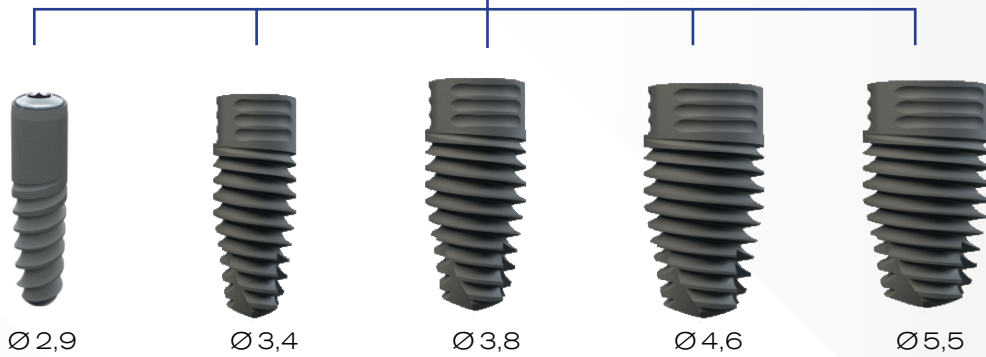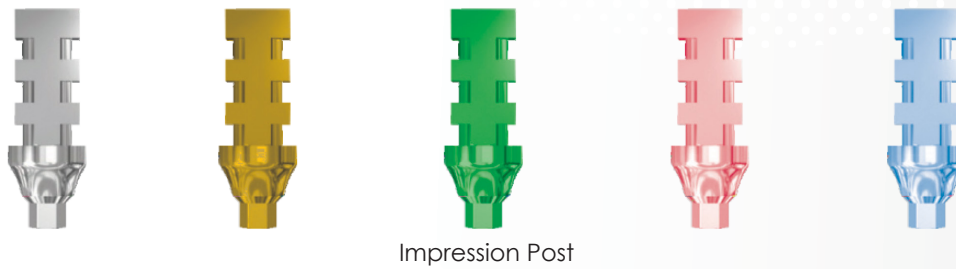
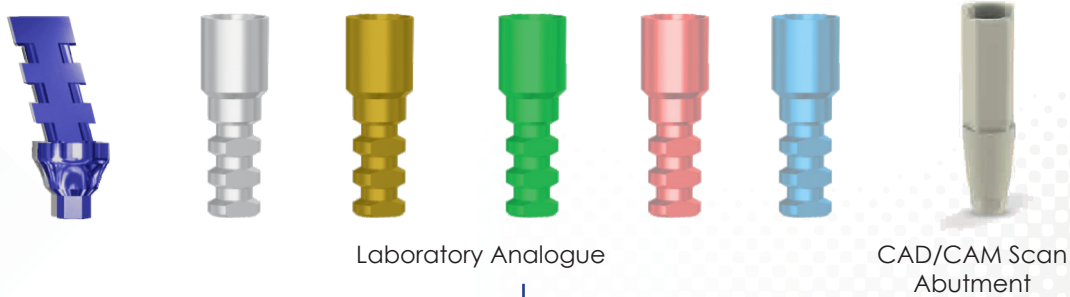
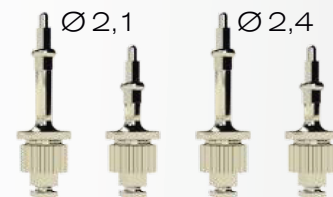
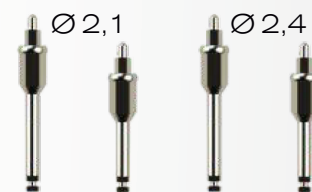
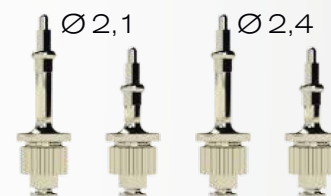
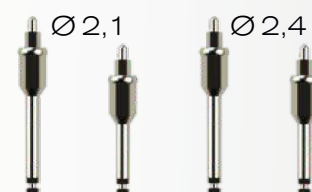
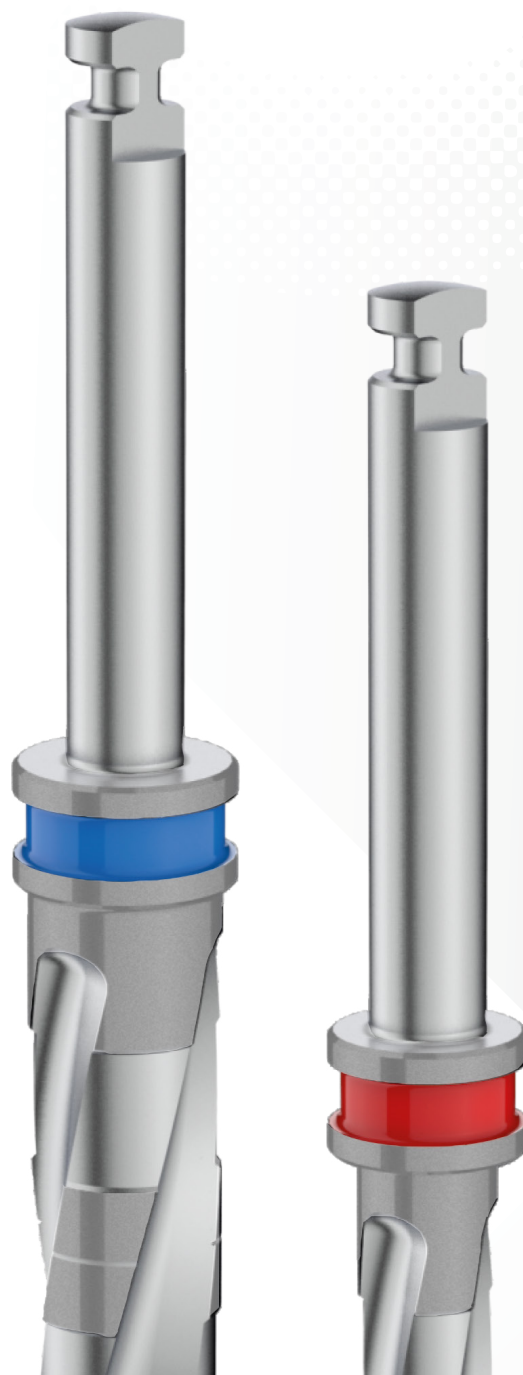
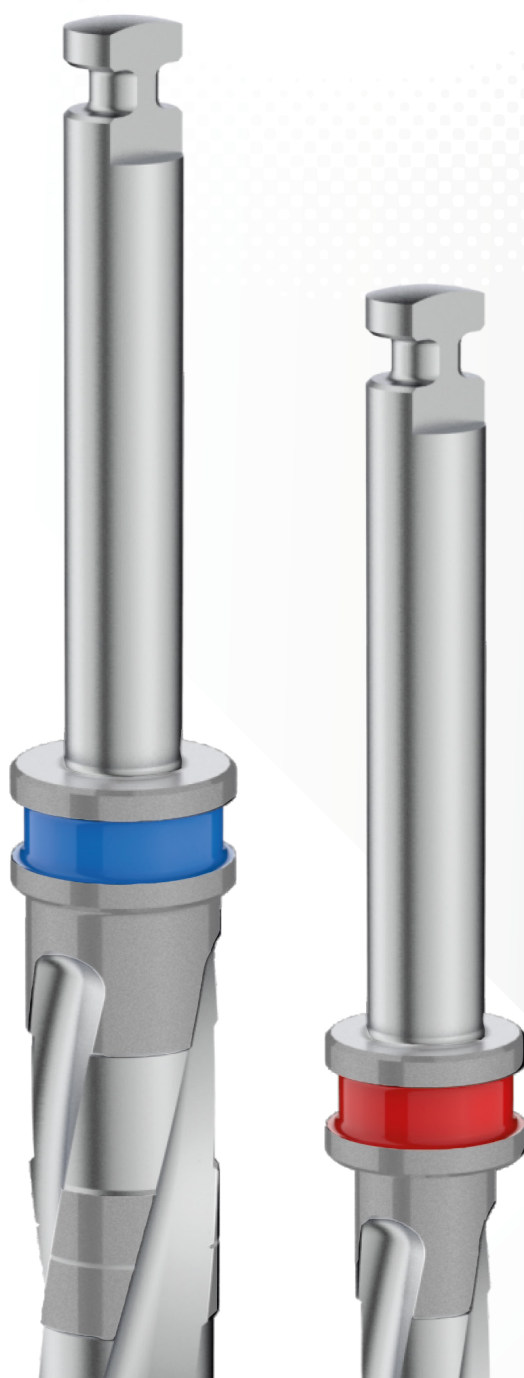
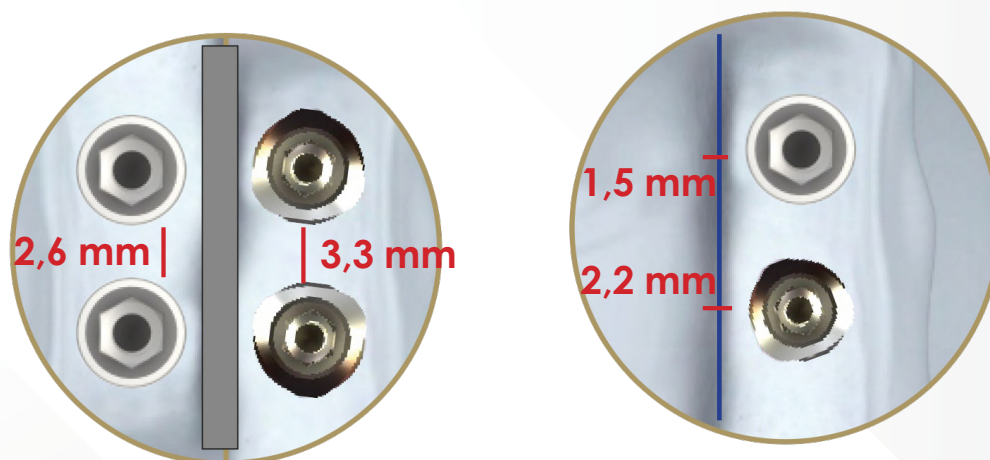
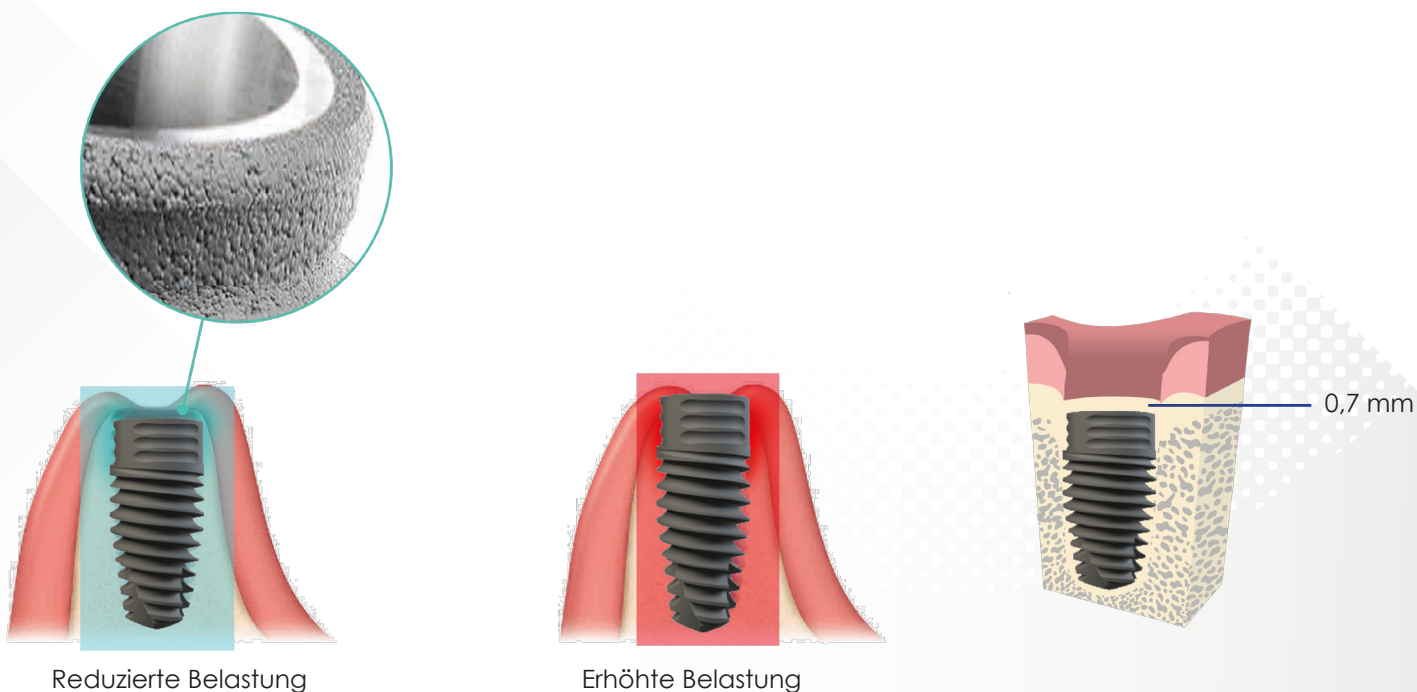
IMPLANT APPLICATION

One of the superior features of the **PRIME** implant is that the implant neck is straight, so the implant cavity depth can be easily controlled. It is recommended to place the implant 0.7 mm below the bone level.

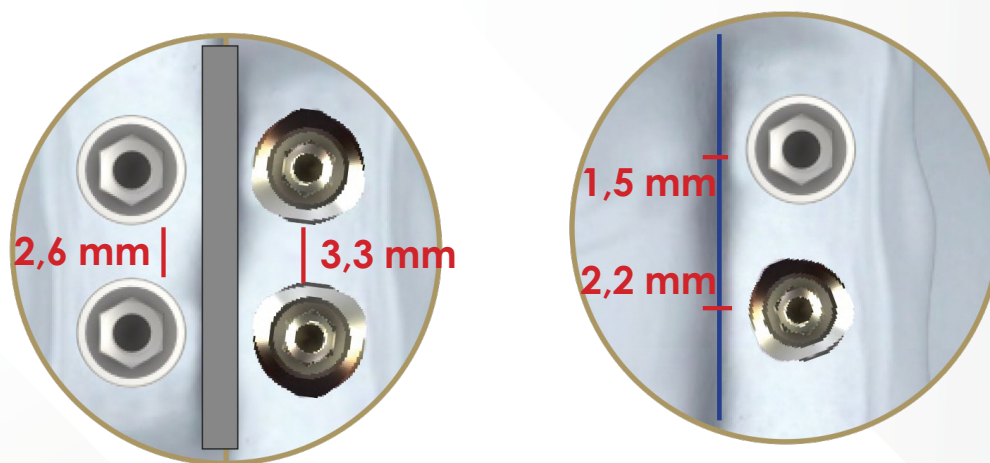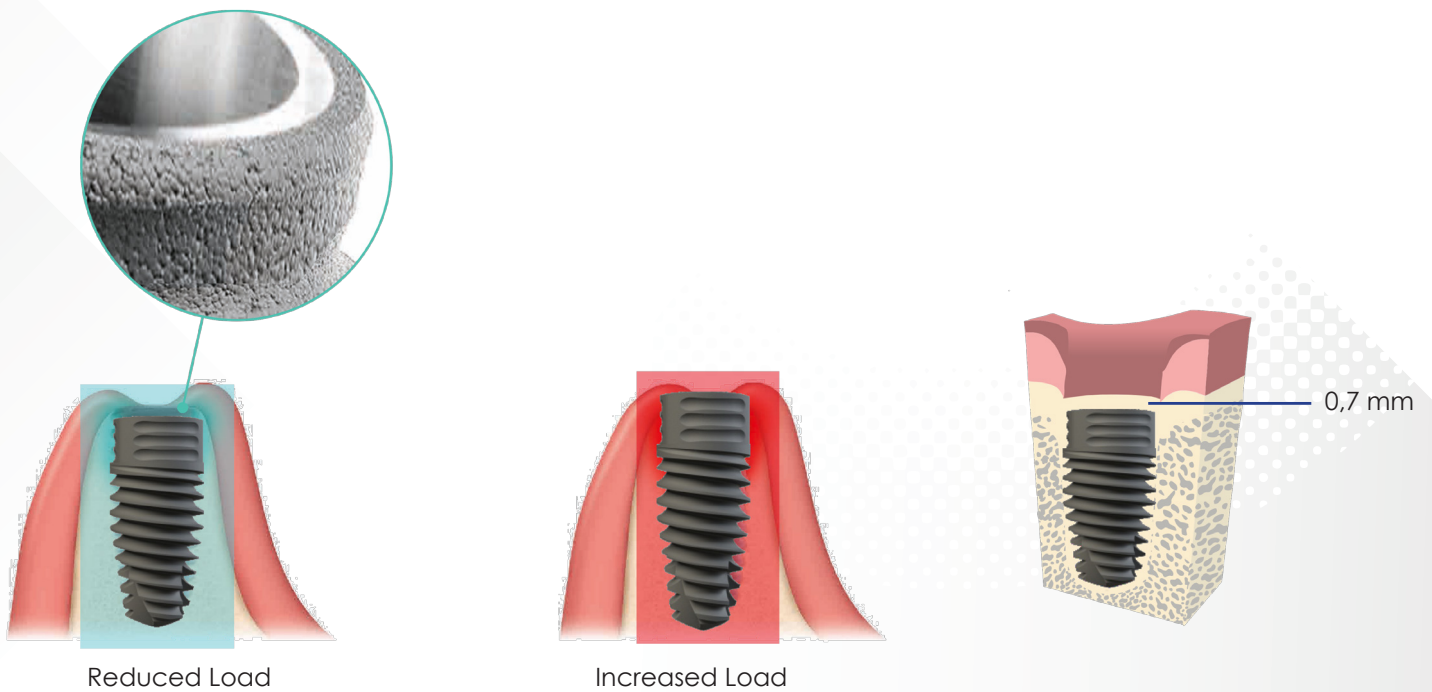Reducing the Load on the Cortical Bone

Designed for more osseointegration space.

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